



We recommend following these tips on how to put on and take off your face shields:

How to adjust & wear face shield:

- ⟨ Wash your hands with soap and water (or hand sanitiser) before you touch the shield.
- ⟨ Insert and thread head strap (B) through the the visor (A) slots on the inside of the visor (Fig2). If printed make sure printed side is facing down when doing this.
- ⟨ Offer visor to face and wrap head strap around head, taking note of where the strap tab meets the nearest strap slot in relation to head size.
- ⟨ Insert head strap tab into the suited strap slot at an angle, then lock both parts by rotating both ends so that the strap is in line (Fig.4).
- ⟨ Place visor on head and readjust strap if necessary. Find a comfortable and safe wearing position (Fig.5) by moving the strap tab to the other nearest strap slot.
- ⟨ Adjust the visor to face distance spacing (Fig.6) by holding the strap level and moving the visor forward or backwards to a suitable and comfortable position.

How to remove a face shield:

- ⟨ Clean hands with soap and water (or sanitiser).
- ⟨ Lift and remove face shield from the back, do not touch the front of your shield (it may be contaminated). Separate parts (A) and (B) disposing of them into the appropriate recycling bins.
- ⟨ Clean your hands again, using soap and water (or sanitiser).